



AVer ViewCare Release note

Product : AVer ViewCare

SW Language : English , 繁體中文 , 한국어 , ykpaiHCbKa

System Requirements



AVer ViewCare

Supported devices

Windows® PC & laptop (Windows® 7, 8, 10, 11) 64bit

System requirements for Windows® PC & laptop

Intel® Core™ 11th i5 or later
8 GB of RAM
10 GB free disk space for installation
100 Mbps network card


- AVer ViewCare served as the terminal to received AI Event that come from the MD Camera and trigger Alarm in the UI with sound.
- AVer ViewCare is a Standalone Software and is NOT cloud-based; it operates exclusively on an intranet.
- For optimal performance with high-resolution, multi-camera streaming, and loop recording features, an upgrade of your CPU, RAM, and storage is recommended. Please contact our sales representative for more information.

SW version 1.1.2014.0

Release date 2025/11/12

New feature **1. Now AVer ViewCare can be paired with and connected to multiple AVer Room Managers, supporting batch management camera FW and log via AVer Room Management , for convenient camera management.**


Reference Architecture:

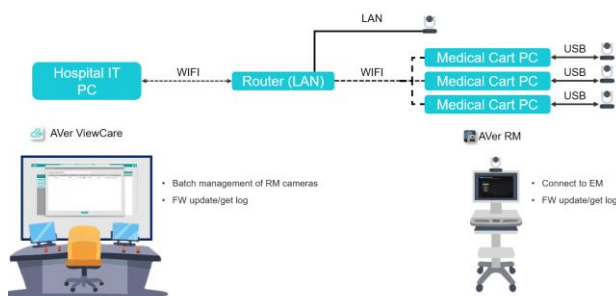


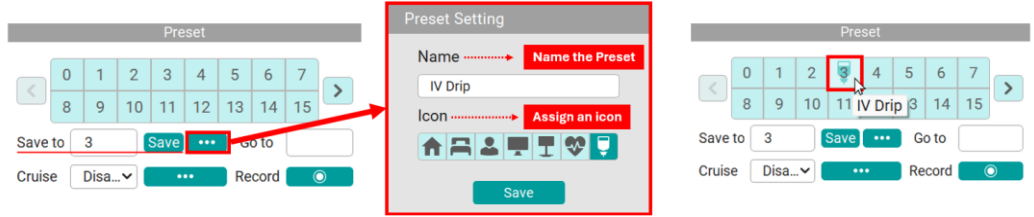
- Room Management V2.3.1019.0 and above are supported.

Improvement **1. Some UI correction**

Known issue	N/A
-------------	-----

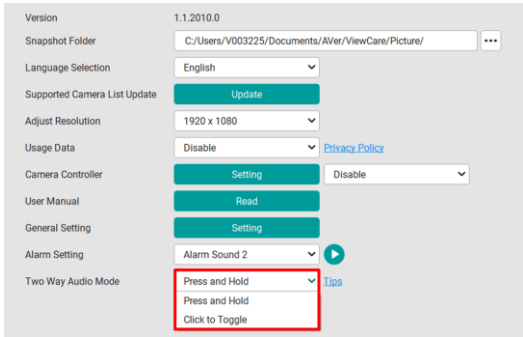SW version	1.1.2010.0
Release date	2025/7/31
New feature	<p>2. Add [Click to Toggle] feature for two-way audio</p> <p>go to [Setup] >> [System] >> change to two-way audio setting</p> <div> <p>[Press and hold]</p> <ul style="list-style-type: none"> - Press and hold the button to start recording and transmitting voice to the camera. - Release the button to stop recording and transmission. <p>[Click to Toggle]</p> <ul style="list-style-type: none"> - Click the button to toggle recording and transmission on or off. - First click starts recording and transmission; second click stops them.  <p>Notice:</p> <ol style="list-style-type: none"> 1. Only support MD330 with FW 1.1.1054.0 and alter 2. 1:1 only , If multiple PC send audio to a single device simultaneously, it may cause sound interruptions or issues 3. If the network is turned off and on again for a short time, it may cause abnormal sound for a few seconds. </div>
Improvement	<ol style="list-style-type: none"> 2. Fixed software may crash when using audio feature on the PC which is not connecting any audio device 3. Now “cancel the alarm” would mute the alarm for 3 minutes 4. Improve recording feature stability 5. Some UI correction
Known issue	N/A

	<p>Example:</p> <p>a Fall alarm occurs in Room 201. after pressing [Cancel and view], the alarm would be mute for 3 minutes and show the live view of the Room 201.</p> <p>Remark:</p> <div><p>AI Detection feature (Fall/Egress) is considered as “the sensor”, so the AI should show the current condition. In other words, when a "fall or leaving the bed" is detected in the screen, an alarm event will continue to be sent, with an interval of every 0.5 seconds.</p><p>Example:</p><ol style="list-style-type: none">when someone fallsAI detects someone has fallen down in FOV it will start to send "fallen event" trigger alarm If the person in FOV does not stand up (still falls down), the camera will continue to issue a "fallen event" every 0.5 seconds.When the person in FOV stands up (no fall detected), the camera will stop sending fall event<p>therefore, [Mute for 3 minutes] in AVer ViewCare can help avoid disturbing users.</p></div>
Improvement	6. Korean UI language correction
Known issue	N/A

SW version	1.1.2005.0
Release date	2024/3/25
New feature	<p>7. Support setting ViewCare with Auto-start, Running in background when closed</p> <div></div> <p>8. You can now assign an icon and rename the preset points in ViewCare Preset name will be displayed when hovering the cursor</p> <div></div>
Improvement	<p>7. ViewCare will now dynamically display the switch based on the AI function supported by the current camera FW.</p> <p>8. Support Recording button in the live view page</p>

Preset

<

0

1

2

3

4

5

6

7

>

8

9

10

11

12

13

14

15

Save to

Save

...

Go to

Cruise

Disa...

...

Record

9. Support Optional Recording Time on Cycle recording

Ex: select 3min , the cycle recording would keep a file in 3 min

Recording

Folder

Hard Disk Info. (GB)

Capacity 0 | Used Space : 0 | Free Space : 0 | Allocated Space: 0

Device Name	IP Address	Mac Address	Storage Size(GB)	Enable	Cyclical Recording Time
MD330UI	10.100.105.112	00:18:1A:09:77:42	0	Disable	<div>5 min</div> <div>1 min</div> <div>3 min</div> <div>5 min</div> <div>10 min</div> <div>No limit</div>

10. Clicking column equal to select the alarm event, to improve operability

Alarm Center | Monitoring

Acoustic alarm

All

Cancel Alarm

History

	Camera Name	Type	Start Time	End Time
120	MD120UI	Fall	2024-03-12 14:14:36	2024-03-12 14:14:44

11. Click [Read] to open the browser to online user manual

Version 1.1.2005.0

Snapshot Folder /Users/V003225/Documents/AVer/ViewCare/Picture/

...

Language Selection

English

Supported Camera List Update

Update

Adjust Resolution

1920 x 1080

Usage Data

Enable

[Privacy Policy](#)

Camera Controller

Setting

Disable

User Manual

Read

General Setting

Setting

12. Fixed some translation errors

13. Fixed the problem of email input restriction

Known issue

N/A

SW version	1.1.2004.0
Release date	2023/10/25
New feature	N/A
Improvement	<ol style="list-style-type: none"> 1. Fixed the icon (shortcut) which auto-generated after installation may not be deleted manually. 2. Fixed in some cases, the last segment of Recording may not be stored. 3. Fixed the issue that might fail to start EyesTracking when more than one man in FOV for some FW version.
Known issue	N/A